

Contact us to receive
our *NEWSLETTER*
and follow us on

Linked in  and  **webpackaging**



ADVANTAGES

The new 120ml ENEMA bottle, with pressure neck closure, is significantly improving compared to previous version (with thread neck closure).

- 1) Reduced number of components from THREE (bottle, cannula, cap), to TWO (bottle, cannula assembled with cover). With this new solution the cap with tamper evident system is not needed anymore.
- 2) The cannula and its cover are automatically assembled as soon as they are printed. The tamper evident system is already included. If needed, the check valve could be assembled automatically.
- 3) The bottle has been designed in order to allow the sticker application.

This final feature will avoid the needs to create the specific mould insertion to print the embossed description on the bottle during the blow moulding process.

USAGE AND APPLICATION

The new ENEMA bottle is made of low density polyethylene (LDPE) in order to obtain a soft and manageable final product. Thanks to this technical solution, the usage is more efficient and the total liquid evacuation is easily performed.

The main applications purpose are:



- 1) Intestine preparation for endoscopic exam.
- 2) Treatment of intestinal constipation.
- 3) Treatment of ulcerative colitis.
- 4) For the whole intestinal diseases that requires rectal applications for better efficient therapy.

The new ENEMA kit is delivered to final users ready to be used. They need just to remove the cover, breaking the tamper evident system and use it without any other operation.

FILLING PROCESS

With this new shape, the filling process is faster and safe. To perform a perfect filling, we suggest to proceed according the following instructions as described:

- 1) Create an appropriate tool able to be inserted under the neck as shown in the picture

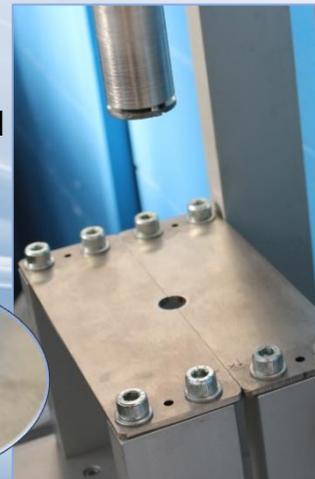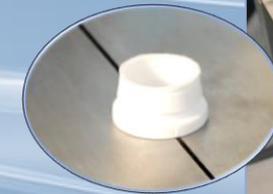

- 2) Proceed with the filling of the liquid.
- 3) Fit the cannula on the neck.

The entire process could be performed automatically.